


EZReader Groups Application

The Groups application allows a user to change various settings, organized into groups of like features.

Clicking on the brief description opens a more detailed description of the Setting. Click on the Setting value changes the Setting to the next available value.

Having like Settings on a single page helps train the user on which Settings are available to affect display and performance features of EZReader.

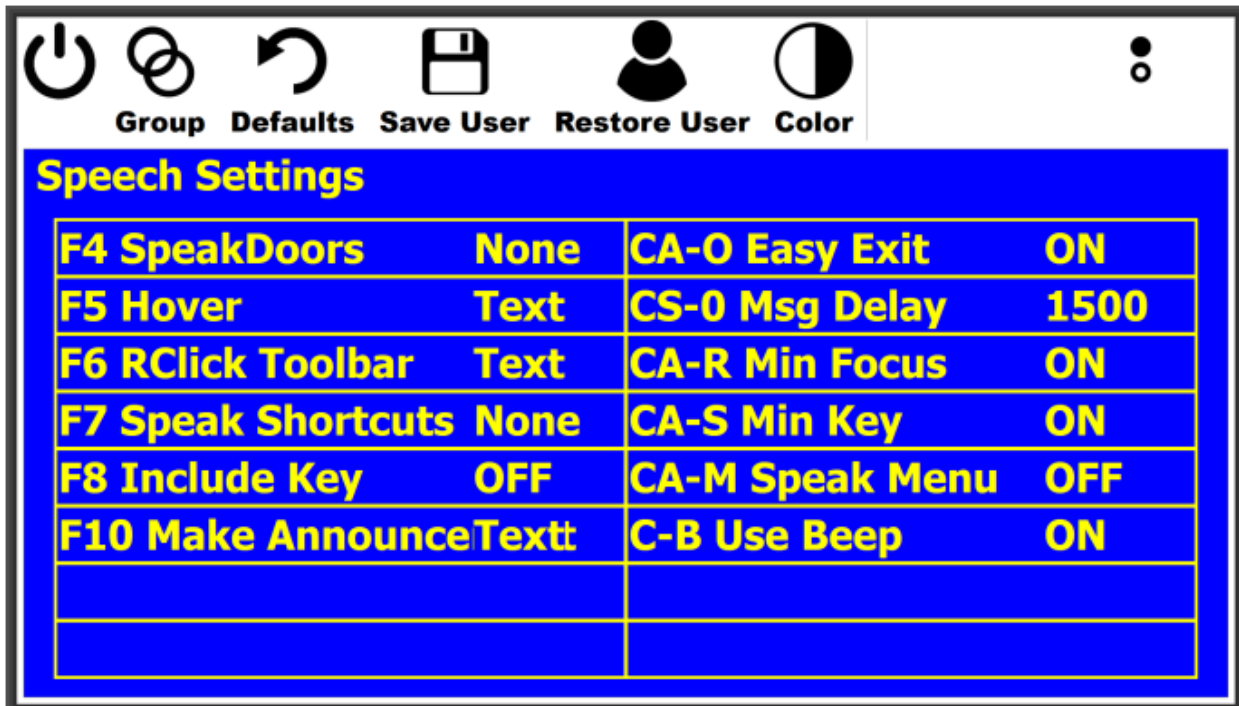

The screenshot displays the EZReader Groups Application interface. At the top, there is a toolbar with six icons: a power button, a group icon, a refresh icon, a save icon, a user icon, and a color icon. Below the icons are labels: "Group", "Defaults", "Save User", "Restore User", and "Color". To the right of these labels is a vertical ellipsis menu icon. Below the toolbar is a blue header bar with the text "Speech Settings" in yellow. Underneath the header is a table with a blue background and yellow text. The table has two columns and six rows. The first five rows contain settings, and the last row is empty.

Speech Settings			
F4 SpeakDoors	None	CA-O Easy Exit	ON
F5 Hover	Text	CS-0 Msg Delay	1500
F6 RClick Toolbar	Text	CA-R Min Focus	ON
F7 Speak Shortcuts	None	CA-S Min Key	ON
F8 Include Key	OFF	CA-M Speak Menu	OFF
F10 Make AnnounceTextt		C-B Use Beep	ON

Toolbar Commands:

Groups has 2 rows of command icons:




Group: Show next group of settings

Defaults: Set current group settings to their default values

Save User: Save current group setting as User-specified values

Restore User: Set current group settings to User-Specified values

Color: Cycle through 4 available color pairs

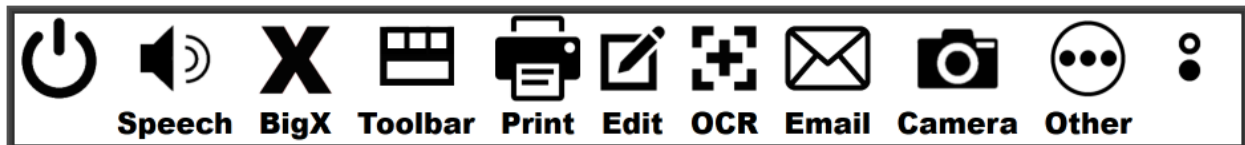

Speech: Show speech settings

BigX: Show BigX settings

Toolbar: Show toolbar settings

Print: Show print settings

Edit: Show edit settings

OCR: Show OCR settings

Email: Show email settings

Camera: Show camera settings

Other: Show other settings